

INSTRUCTIONS FOR USE

Sterile Infusion Set for single use

(With air vent and without air vent)

MATERIAL USED:-

Polyvinyl Chloride (PVC), Acrylonitrile butadiene styrene (ABS), Polypropylene, Latex Bulb/silicon stopper for injection site.

INDICATION:-

Administration of Infusion fluids intravenous with the help of hypodermic device.


Infusion set is use to administrate the fluids from a container to a patient's vascular system, through a needle or catheter inserted in to a vein under gravity feed and medical application in order to ensure their compatibility for infusion solution.

CONTRAINDICATION:-

1. Device must not be used other than as indicated in intended use.
2. Device should not be used for administration of blood and blood components.
3. Device should not be used in patients with known hypersensitivity to any of the materials used.
4. Device should not be used for high viscous fluid.

INSTRUCTION FOR USE:-


1. Inspect the package for proper sealing integrity to make sure that device is sterile and if individual package has been opened or tampered, please do not use.
2. Remove the Infusion Set from package using aseptic technique.
3. Select site for vein puncture. Apply tourniquet and prepare site with appropriate antiseptic.
4. Perform vein puncture with patient's arm in downward position and assure that the infusion set is securely attached to the patient.

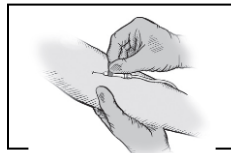

5. To achieve the required flow. The flow of infusion fluid to be regulating as per requirement by using the provided regulator in Infusion set.

INSTRUCTIONS FOR USE

- Please ensure that the regulator of infusion set shall be off position, during the changing of infusion fluid container.
- After completion of infusion, hold the gauze down without applying pressure.

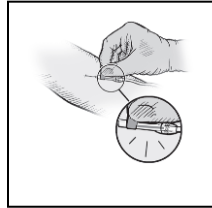

- Promptly dispose of Infusion Set in an approved disposal container in accordance with the procedures of your facility.
- Single use only, does not re-sterilize, Infusion set should be discarded after one procedure. It's extremely difficult to clean exactly after being expressed to biological materials and may causes adverse patient reactions if reuse, the cleaning of these devices may alter their structural property accordingly Lifelong Meditech will not be responsible, direct incidental and consecution damages resulting for reuse.

WARNINGS: -

- Device is single use only.
- Device must not be reused. Reuse of the device may ;**
 - Cause serious infection, cross contamination, transmission of bacteria, viruses and blood related transfusion of serious diseases like HIV, Hepatitis.
 - Alter the performance & function like sharpness of piercing device, alter of flow regulator.
- Ensure before use that device/packing is not tempered or damage.
- Read instruction before use.
- Check expiry date prior to use.
- The device should be used according to the instruction for use.
- Use device immediately after opening of individual pack.
- This device should be used by a Doctor/Registered Practitioner or Paramedic.
- Discard all used sets together with the holder in biohazard containers approved for their disposal.
- Make sure all air is removed by priming prior to use as short-term IV infusion device.
- The device will perform as intended when the instructions are followed accordingly.
- Examine individual package for integrity prior to use. If packaging has been damaged, do not use.

INSTRUCTIONS FOR USE

13. Do not use for subcutaneous infusion or injection.
14. Avoid leakage and any air in the tubing during infusion procedure.
15. Please do not use by/on those individuals which are sensitive/allergic to phthalate.
16. The Phthalate (DEHP) may be present in device which needs medical attention while using to pregnant or nursing women and children.
17. The latex may be present in device which needs medical attention to patients/user.
18. Improper transport and mishandling may cause structural and / or functional damage to device or packaging.

STORAGE OF INFUSION SETS BEFORE USE







Store the Infusion Set at cool, dry and dark places.

NOTE: Avoid exposure to direct sunlight. Exceeding the maximum recommended storage temperature may lead to impairment of the Infusion Set quality.

















DISPOSAL:-

After use, dispose of device and packaging in accordance with hospital, administrative and/or local government policy.



LABEL INFORMATIONS:-

Symbol	Description
	Catalogue Number
	Lot Number: Batch Number
	Expiry date: Device can be used until the end of the month indicated
	Sterilization with ethylene oxide gas
	single use only
	Manufacturer

INSTRUCTIONS FOR USE

	Warnings
	Conformité Européenne (CE marking is a mandatory conformity marking for certain devices sold within the European Economic Area)
	Do Not Resterilize
	Non Pyrogenic
	Do Not Use if Package is Damaged
	Keep Dry
	Keep Away From Sunlight
	Date of Manufacture
	Authorized Representative In The European Community
	Consult Instruction For Use
	Sterile Fluid Path
	Fluid Filter Pore size
	Contains or presence of Phthalate
	Contains or presence of natural rubber latex
	Symbol for 20 ml = 1ml ±0.1ml (Distilled water)
	Do Not use hook

INSTRUCTIONS FOR USE

	Store and stack in upright condition
	Fragile



Lifelong Meditech Private Limited,
Plot No. 18, Sector – 5, IMT Manesar,
Gurgaon – 122050, Haryana (INDIA),
Mfg. Lic. No. 709-B(H),
lmlcustomercare@lifelongindia.com
Customer Care No.: +91-124-4406600



MED DEVICES LIFE SCIENCES LTD.
145 -157, St. John Street, EC IV 4PY, LONDON,
U.K. Tel:+44 84552 80533,
Email: info@meddevices.net