
1.0 INSTRUCTION FOR USE

1.1 PRODUCTS

- 1.1.1 **Sterile Hypodermic single use Syringe with/without Needles** (1ml, 2ml, 3ml, 5ml, 10ml, 20ml and 50ml).

- 1.2 **MATERIAL USED:** - Polypropylene, Thermoplastic Elastomer, SS304.

- 1.3 **INDICATIONS:** - Administrations of injections or medicinal fluids intra muscular and intra venous. Sterile hypodermic single use Syringes with needles, intended for injection of intravenous and/or intra muscular fluids immediately after filling.

The role of medical devices is for treatment and alleviation of disease. The devices are use for injecting the medicinal fluids as fixed dosage in the body.

1.4 CONTRAINDICATIONS:-

- 1.4.1 Device must not be used other than as indicated in intended use.
- 1.4.2 Device should not be used for administration of blood and blood components.
- 1.4.3 Device should not be used for large volume fluids and fluids derivatives.
- 1.4.4 Device should not be used for high viscous fluid.

1.5 INSTRUCTION FOR USE:-

- 1.5.1 Inspect package for integrity and expiry and then remove the syringe from package using aseptic technique.
- 1.5.2 Select proper size of syringe & needle for injection.
- 1.5.3 Carefully select and clean the venipuncture or muscle site.
- 1.5.4 Be careful to pull the needle cap straight off to avoid the needle point damage.
- 1.5.5 Push and twist the needle clock wise on the syringe tip to seal it firmly and ensure a tight fitting.
- 1.5.6 Expel, air bubble formation through open end before injection.
- 1.5.7 Cover the puncture site with a sterile dressing-
- 1.5.8 Device is single use only. Do not re-sterile. Syringe should be discarded after one procedure. It's extremely difficult to clean exactly after being expressed to biological materials and may causes adverse patient reactions if reuse, the cleaning of these devices may alter their structural property accordingly lifelong Meditech will not be responsible for direct incidental and consecution damages resulting for reuse.

1.6 WARNINGS:-





- 1.6.1 Device is single use only.
- 1.6.2 **Device must not be reused. Reuse of the device may;**













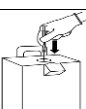



- 1.6.2.1 Cause serious infection, cross contamination, transmission of bacteria, viruses and blood related transfusion of serious diseases like HIV, Hepatitis.
- 1.6.2.2 Alter the performance & function like sharpness of needle, smoothness of plunger movement and needle tube fluidity.
- 1.6.3 Ensure before use that device/packing is not tempered or damage.
- 1.6.4 Read instruction before use.
- 1.6.5 Check expiry date prior to use.
- 1.6.6 The device should be use according to the instruction for use.
- 1.6.7 Use the device immediately after opening the individual pack.
- 1.6.8 This device should be used by a Doctor/Registered Practitioner or Paramedic.
- 1.6.9 Avoid any air bubble in the syringe during infusion procedure.
- 1.6.10 Do not re-shield used needles.
- 1.6.11 Gloves should be worn at all times during vein puncture to minimize exposure hazard.
- 1.6.12 Caution must be taken when collecting blood samples from immobilized, hemophilic or epileptic patients.
- 1.6.13 Due to the risk of surface contamination from residual blood that may be on the needle tip, it is recommended to dispose of the needle with the tip directed upwards.
- 1.6.14 Discard after use. Do not re-cycle, clean or re- sterile.
- 1.6.15 Improper transport and mishandling may cause structural and / or functional damage to device or packaging.

1.7 **STORAGE:** - Store in a cool and dry place.

1.8 **DISPOSAL:** - After use, dispose of device and packaging in accordance with hospital, administrative and/or local government policy.

1.9 **LABEL INFORMATIONS:-**

Symbol	Description
R_x Only	Device to be sale on order/prescription of a Physician/Doctor/Paramedical staff only
	Catalogue Number
	Lot Number: Batch Number
	Expiry date: Device can be used until the end of the month indicated
	Sterilization with ethylene oxide gas

	single use only
	Manufacturer
	Warnings
	Conformité Européenne (CE marking is a mandatory conformity marking for certain devices sold within the European Economic Area)
	Do Not Resterilize
	Non Pyrogenic
	Do Not Use if Package is Damaged
	Keep Dry
	Keep Away From Sunlight
	Date of Manufacture
	Authorized Representative In The European Community
	Consult Instruction For Use
	Safe Disposal of syringe
	Do Not use hook
	Store and stack in upright condition
	Fragile

 **Lifelong Meditech Private Limited,**
Plot No. 18, Sector – 5, IMT Manesar,
Gurgaon – 122050, Haryana (INDIA),
Mfg. License No. MFG/MD/2019/000203
lm1customer@lifelongindia.com
Customer Care No: +91-124-4406600

 **MED DEVICES LIFESCIENCES B.V.**
Kraijenhoffstraat 137 A, 1018RG Amsterdam, The
Netherlands.
Tel : +31 202254558
Email: info@meddevices.net